

dermadrive™

Dermatome skin grafting system

THE NEW
GD SERIES




deSoutter
MEDICAL

GD PRECISION SKIN GRAFTING SYSTEM

The new slimline and ergonomically designed *GD Series* dermatome from De Soutter Medical produces consistent, uniform skin grafts time after time. The smooth high speed cutting action ensures an almost effortless and vibration free procedure. The GD-113 dermatome head is designed to be used with the CM2 controller system. Alternatively, the head with its integral E coupling allows it to be used with other De Soutter handpieces, for instance, EPV pneumatic and ECV electric motors. Please ask for further details.



Cut thicknesses are easily selected and constantly maintained using the calibrated depth gauge. Skin grafts from 0.05 mm to 1.00 mm are adjustable in 0.1 mm increments.

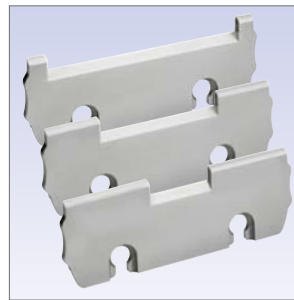

Accurately controlled graft widths of 73 mm, 48 mm and 23 mm are readily achieved by fitting the appropriate easy-fit sole plate.




Stericut precision ground, double facet, ultra sharp blades provide accurate and consistent cutting performance whilst permitting very thin skin grafting. Blades are supplied sterile packed (5 per box). The GD-113 dermatome head enables quick and safe blade replacement.



SYSTEM SPECIFICATION

DERMATOME SPECIFICATION

	Cutting width mm	Cutting thickness range mm	Operating speed range rpm	Weight kg
	73	0.05 - 1.00	2000 - 15000	0.28
The dermatome head is supplied with a 73 mm sole plate as standard. Optional sole plates: 48 mm cutting width - Part No. 17260 23 mm cutting width - Part No. 17340				

MICRO MOTOR AND POWER SUPPLY

The dermatome head fits quickly onto the universal micro motor which is powered by the variable speed CM2 controller. The motor can be controlled either by the footswitch or directly from the controller itself.

Please specify voltage and plug configuration when ordering.

Power supply specification

Input voltage: 230 (120) VAC/50 (60) Hz
Power: 130VA
Dimensions: 100 x 85 x 165 mm
Weight: 1.68/0.12 kg controller/motor



STERILISATION CONTAINER

This purpose designed compatible delivery system will ensure your instruments are maintained in good condition at all times. The container is 100% sterilisable by steam, dry heat ('flash') to 160C, ethylene oxide (ETO) or gamma irradiation.



HOW TO ORDER

System components	230V - UK plug	230V - Euro plug	120V - US plug
GD-113 head complete	17200	17200	17200
CM2 motor controller and footswitch	1280884	1280994	1281004
Blades (pack of 5)	S11-103	S11-103	S11-103
Sterilisation container	8570	8570	8570
Complete system ordering code	GDA-113/230UK	GDA-113/230EU	GDA-113/120US

***For further information on De Soutter Medical
powered instrument and cast saw systems
please contact the address in your area***

UNITED KINGDOM (INTERNATIONAL HEAD OFFICE)

De Soutter Medical Limited

Halton Brook Business Park, Weston Road, Aston Clinton, Aylesbury, Bucks HP22 5WF

Tel: +44 (0) 1296 634 000

Fax: +44 (0) 1296 634 033

Email: info@de-soutter.com

AUSTRALIA

De Soutter Medical Australia Pty Ltd

2/12-14 Apollo Drive, Hallam, Victoria 3803

Tel: +61 (0) 3 9702 4441

Fax: +61 (0) 3 9702 4484

Email: australia@de-soutter.com

AUSTRIA

De Soutter Medical Austria

Zweigniederlassung Österreich, Dietrichsteingasse 10, A-3400 Klosterneuburg

Tel: +43 (0) 676 96 71 770

Fax: +43 (0) 2243 21 656

Email: austria@de-soutter.com

BELGIUM

De Soutter Medical Belgium

In De Bruel 30, 3620 Lanaken

Tel: +32 (0) 89/47 15 37

Fax: +32 (0) 89/70 12 19

Email: belgium@de-soutter.com

FRANCE

De Soutter Medical France

949 Avenue Parc des Expositions, 33260 La Teste de Buch

Tel: +33 (0) 5 56 54 89 36

Fax: +33 (0) 9 70 61 37 60

Email: france@de-soutter.com

GERMANY

De Soutter Medical Germany

Niederlassung Deutschland, Kladenfloss, D-66625 Nohfelden

Tel: +49 (0) 68 52-99 12 46

Fax: +49 (0) 68 52-99 12 47

Email: deutschland@de-soutter.com

ITALY

De Soutter Medical Italia

Palazzo Marco Polo, Il Girasole, 20084 Lacchiarella, Milano, Italia

Tel: +39 (0) 2 9009 4098

Fax: +39 (0) 2 9009 2673

Email: italy@de-soutter.com

www.de-soutter.com

120-EN-5